




QuikRead go[®] HbA1c

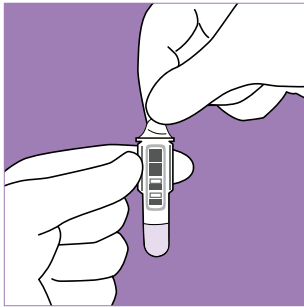


1 Remove the foil cover from the cuvette. Do not touch the clear flat surfaces in the lower part of the cuvette.

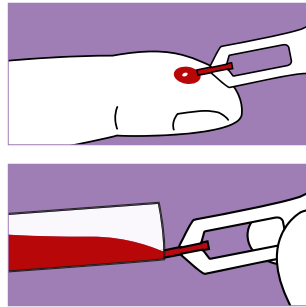


2 Hold QuikRead go Sample Collector 1µl almost horizontally just beneath the surface of the sample. Fill the capillary part of the sample collector completely.

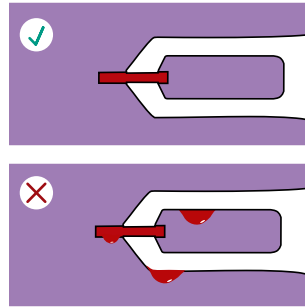


Note! Check that the capillary part is completely filled. **Do not wipe the sample collector!** If there is excess sample on the sample collector, discard the sample collector.

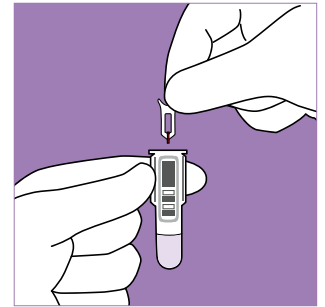

3 Insert the sample collector into the cuvette within 1 minute. The sample collector does not touch the solution in the cuvette.

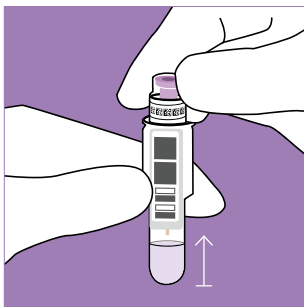

4 Close the cuvette tightly with the HbA1c reagent cap. Do not press down the violet inner part of the reagent cap. Perform the test within 5 minutes. **Keep the cuvette upright and do not shake.**

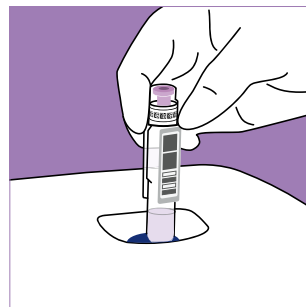

5 Choose **Measure** on the display of the QuikRead go instrument. Insert the cuvette into the measurement well of the instrument. The barcode should be facing you.



6 The result appears on the display and the cuvette rises up automatically when the measurement is completed.

Please consult the instructions for use before performing the test.

QuikRead go[®] HbA1c



Kit

Unopened kit at 2–8°C: until **the expiry date**

Unopened kit at 18–25°C: **2 months**



Cuvettes

Unopened foil pouch at 2–8°C: until the **expiry date** of the kit

Unopened cuvettes after opening the foil pouch:

at 2–8°C: **6 months** / at 18–25°C: **2 months**



Opened cuvette

Let the buffer warm up to 18–25°C before testing.

Perform the test within **1 hour** from opening the cuvette.

The sample is stable in the cuvette for **5 minutes** with the closed cap.



Reagent cap
tube

Unopened tube at 2–8°C: until the **expiry date** of the kit

After the first opening at 2–8°C: **6 months** / at 18–25°C: **2 months**



The sample
collectors

Storage at 2–25°C:

until the **expiry date** of the Sample Collector 1µl box



HbA1c
Control Set

Unopened vial at 2–8°C: until the **expiry date**

Opened vial at 2–8°C: up to **50 days**

Contact info

Aidian Oy

aidian@aidian.eu

+358 10 309 3000

For orders: orders@aidian.eu

www.aidian.eu

www.quikread.com

Product	Cat. no.
QuikRead go HbA1c with QuikRead go Sample Collector 1µl, 25 tests	151058
QuikRead go HbA1c Control Set	154520
QuikRead go Instrument	133893
QuikRead go Sample Collector 1µl, 25 pcs	154457